



Identity.net User Manual

Version 4.0

2023-03-29

Contents

- 1. Identity.Net3**
 - 1.1. Introduction..... 3
 - 1.2. Module selection..... 4
 - 1.3. Association procedure..... 5
 - 1.4. Disassociation procedure 6
 - 1.5. Annex – Examples of user workflows..... 7
 - 1.5.1. Application selection 7
 - 1.5.2. Patient – Device association procedure 7
 - 1.5.3. Disassociation procedure..... 7

1. Identity.Net



For general and detailed information about the Product environment and the instructions for use of the Control Bar software, see the specific documents of the Product. The knowledge and understanding of these documents is mandatory for an appropriate and safe use of the Identity.Net module, described in this document.

1.1. Introduction

The Identity.Net module makes it possible to quickly associate/disassociate medical devices to patients.



Only a sub set of the configured devices is eligible for the quick association / disassociation procedure. Usually the procedure applies to wi-fi devices that are often moved from one patient to another. In all cases, it is the features, mode of use and configuration of the device that makes it or not eligible for the procedure here described. Only the eligible devices are displayed on the Identity.net module.



If wireless infusion pumps are in use, these are automatically disconnected from the patient when out of Wi-Fi coverage or powered off for more than the number of seconds specified in the configuration option "PatientDeviceAssocTimeout".



If wireless infusion pumps are in use, it is necessary to associate them to the patient every time a new infusion is started.



If a patient is discharged and then readmitted, the devices remain associated to the previous patient admission. Use the Identity module to associate the devices to the most recent patient admission.

1.2. Module selection

To access the Identity module on desktop workstations:

- Click the  icon.

The following screen is displayed (Fig 1).

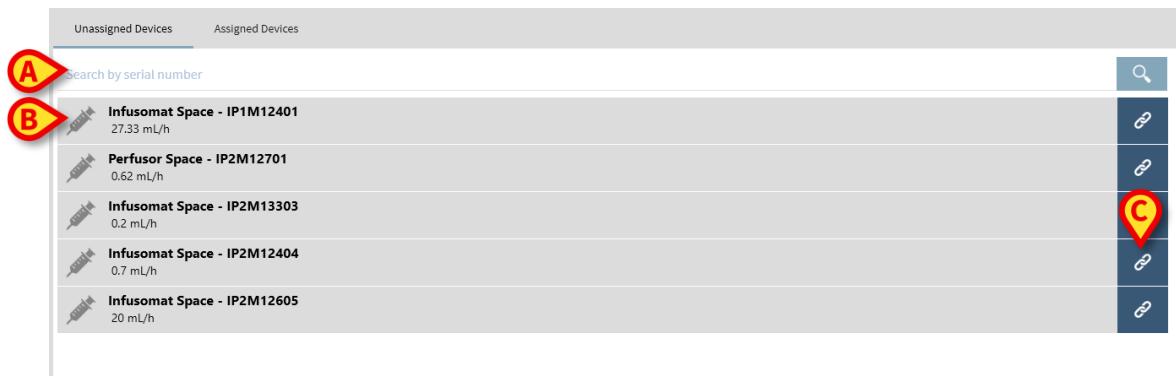


Fig 1

Two tabs, indicated in Fig 1 **A**, allow to select either the list of devices associated to a patient (“Assigned”) or the list of devices not assigned to a patient (“Unassigned”). Default is “Unassigned”.

1.3. Association procedure

To assign a device to a patient,

- Select the “Unassigned” tab (if not selected already).

The list of not assigned devices is this way displayed. Each row corresponds to a device.

- Find the device to be assigned.

A search tool is available (Fig 1 **B**). You can search by device serial number.

Search by barcode scan is also available. Scan the wanted device barcode to display a row referring to the related device.

On the row corresponding to the wanted device,

- Click the icon indicated in Fig 1 **C**.

The following screen opens (Fig 2), requiring to select the patient to whom the device will be assigned.

Assign

Location: ICU

Bed: 2 - Patient Bill - Code: 456344

Do you confirm the association of patient:

Patient, Bill ♂
Code: 456344 Birthdate: 12/10/1977
Admitted in ICU - Bed 2

with medical device:

Infusomat Space - IP2M13303
0.2 mL/h

✓ Assign ✕ Cancel

Fig 2

- Use the drop down lists indicated in Fig 2 **A** to select the patient.

The selected patient and selected device data is displayed on the window (Fig 2 **B**).
If data is correct, then

- click the button indicated in Fig 2 **C**.

The device-patient association is this way completed.

1.4. Disassociation procedure

To disassociate a device from a patient.

- Access the Identity module.
- Select the “Assigned” tab (Fig 3 **A**).

The following content is displayed.



Fig 3

Dark-grey rows refer to patients (Fig 3 **B**). Light-grey rows refer to devices (Fig 3 **C**).

All the devices assigned to a patient are listed below the patient name. A search tool is available (Fig 3 **D**), making it possible to search for a specific device. Search can be performed by device serial number/patient name/patient code.

- Find the row corresponding to the device to be disassociated.
- Click the  button (Fig 3 **E**). The following window opens, requesting user confirmation (Fig 4).

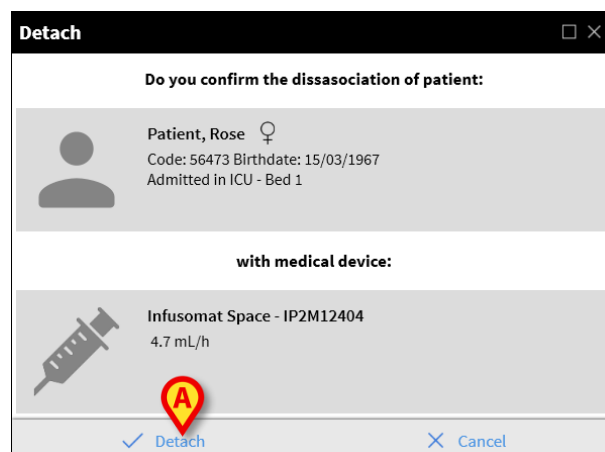


Fig 4

- Click the button indicated in Fig 4 **A** to complete the disassociation procedure.

1.5. Annex – Examples of user workflows

1.5.1. Application selection

To access the Identity module on desktop workstations:

- 1 Click the  icon on the lateral bar. The screen shown in Fig 1 is displayed.

The screen lists the unassigned devices. Each row corresponds to a device (Fig 1 **A**).

1.5.2. Patient – Device association procedure

To assign a device to a patient,

- 2 Find the device to be assigned. A search tool is available (Fig 1 **B**).

On the row corresponding to the wanted device,

- 3 Click the icon indicated in Fig 1 **C**.

The screen shown in Fig 2 opens, requiring to select the patient to whom the device will be assigned.

- 4 Use the drop down lists indicated in Fig 2 **D** to select the patient.

The selected patient and selected device data is displayed on the window Fig 2 **E**.

- 5 Click the **Assign** button indicated in Fig 2 **F**.

This completes the device-patient association.

1.5.3. Disassociation procedure

To disassociate a device from a patient.

- 1 Select the 'Assigned Devices' tab (Fig 3 **A**).

Dark gray rows refer to patients. Light gray rows refer to devices. All the devices assigned to a patient are listed below the patient name. A search tool is available (Fig 3 **B**), making it possible to search for a specific device.

- 2 Find the row corresponding to the device to be disassociated.

- 3 Click the  button (Fig 3 **C**).

The window shown in Fig 4 opens, requesting user confirmation.

- 4 Click **Detach** (Fig 4 **D**) to complete the disassociation procedure.

Unassigned Devices	Assigned Devices
Search by serial number	
B Alaris neXus PK - IP2M12402 26 mL/h	C
Alaris neXus PK - IP2M13303 0.62 mL/h	
Alaris neXus PK - IP2M12404 24 mL/h	
A Alaris neXus PK - IP2M12605 13.33 mL/h	
Alaris neXus PK - IP3M13202 26 mL/h	
Alaris neXus CC - IP4M12902 0.34 mL/h	
Alaris neXus CC - IP4M12703 9 mL/h	
Alaris neXus CC - IP4M12304 3.6 mL/h	
Alaris neXus CC - IP4M12305 16 mL/h	

Fig 1

D

Location: ICU

4 - John JD Doe - Code: 34526

Do you confirm the association of patient:

John, Doe JD ♂

Code: 34526 Birthdate: 01/01/1954

Admitted in ICU - Bed 4

With device:

Alaris neXus CC - IP2M12701

Gabexate mesylate 4.2 mL/h

F

Assign Cancel

Fig 2

Unassigned Devices	Assigned Devices
Search by device serial number, patient name, patient code	
B John, Doe JD ♂	C
Alaris neXus CC - IP2M12701 2.2 mL/h	
Alaris neXus PK - IP2M13303 26.67 mL/h	
John, Smith JS ♂	
Code: 45367 Birthdate: 01/01/1952	
Alaris neXus PK - IP2M12404 Diltiazem 30.67 mL/h	

Fig 3

D

Do you confirm the disassociation of the patient?:

John, Doe JD ♂

Code: 34526 Birthdate: 01/01/1954

Admitted in ICU - Bed 4

With device:

Alaris neXus PK - IP2M13303

30 mL/h

D

Detach Cancel

Fig 4