



Digistat V6.0

Quick Reference Guide

Revision 1.0

2019-06-11

Ascom UMS s.r.l. Unipersonale

Via Amilcare Ponchielli 29, IT-50018 Scandicci (FI), Italia

Puh. (+39) 055 0512161 – Fax (+39) 055 829030

www.ascom.com

Content

- 1. Start-up procedure..... 3**
 - 1.1 Digistat installation..... 3
 - 1.2 How to run Digistat..... 4
 - 1.3 Log In procedure 5
 - 1.3.1 Barcode login 6
 - 1.3.2 How to disable the automatic user log out (“User lock” functionality) 6
 - 1.4 Log out 6
- 2. System unavailability 7**
- 3. Digistat Instructions for Use 8**
 - 3.1 On line update of the user manual 8
- 4. How to display the Digistat about box..... 9**
 - 4.1 Printed Instructions for Use10
- 5. Contacts 11**


1. Start-up procedure

1.1 Digistat installation

The Digistat system can only be installed by Ascom UMS technicians or technicians explicitly authorized by Ascom UMS. The end-user will find the system already installed and properly configured on his/her workstation.

1.2 How to run Digistat

To run Digistat:

- Double click the  icon on the workstation desktop.

The following window is displayed while the system is loading (Fig 1).



Fig 1

After a few seconds the Digistat work area is displayed (Fig 2).

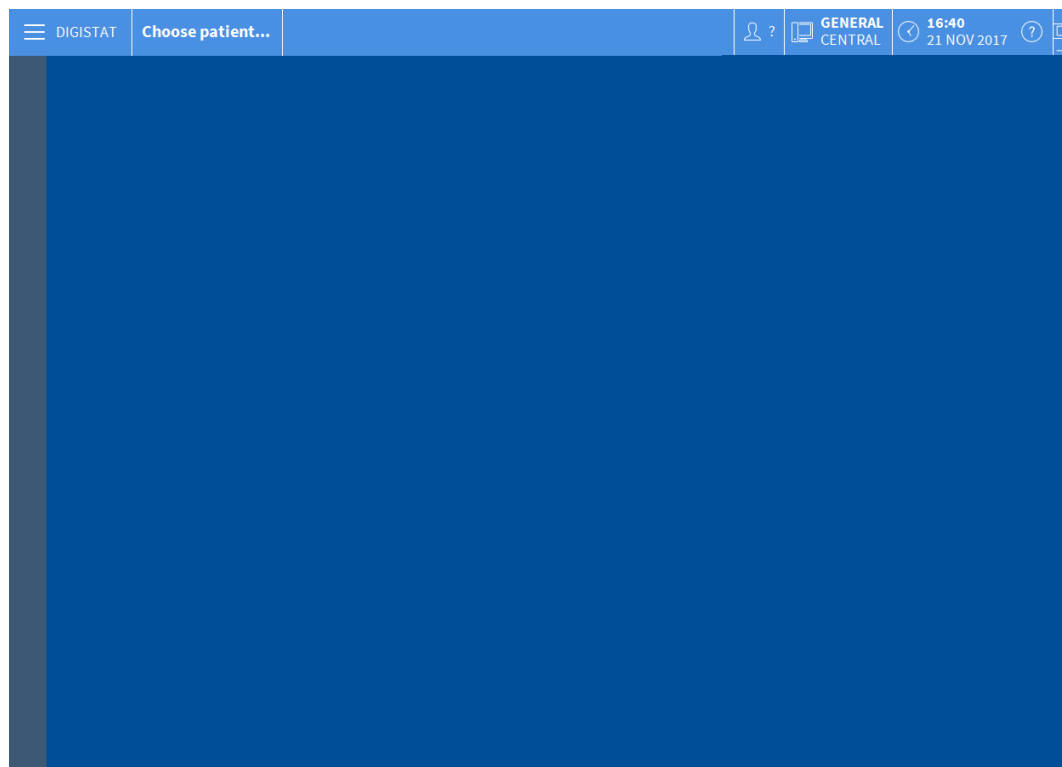


Fig 2

1.3 Log In procedure

To use Digistat it is necessary to log in to the system. The user must insert his/her credentials (user name and password) at the beginning of every work session.

- Click the **User** button indicated in Fig 3 **A**. The following screen is displayed.

Fig 3

Then, on the screen:

- Insert your username in the “Username” field (Fig 3 **B**).
- Insert your password in the “Password” field (Fig 3 **C**).
- Click the **Ok** button (Fig 3 **D**).

The user is now logged in. An acronym indicating the user currently logged is displayed on the **User** button on the control bar (the acronym is “ADM” in Fig 4).



Fig 4

1.3.1 Barcode login

It is possible to log in through barcode scanning, where the healthcare organization uses this technology.

To use this functionality, when the system displays the login screen (Fig 3),

- scan the user's personal barcode.

If the barcode is recognized the user is immediately logged in.

1.3.2 How to disable the automatic user log out (“User lock” functionality)

If the system remains idle for a certain time (configurable), the user is automatically logged out.

To disable this functionality it must be done, after username and password specification but before clicking **Ok**,

- Click the **Lock** button on the “Log in” screen command bar (Fig 5 **A**).



Fig 5

If the user is locked, a padlock icon is shown on the user button (Fig 6).



Fig 6

1.4 Log out

To close a work session with Digistat, the user must “log out” from the system.

To log out:

- Click the **User** button on the command bar (Fig 7 and Fig 3 **A**).



Fig 7

The user acronym disappears from the **User** button.

2. System unavailability

If, during start up, there are problems connecting with the server, the system provides a specific “System recovery” window.

The connection problem sometimes is automatically solved after a short time. If not, please contact Ascom UMS technical assistance. See section 5 for Ascom UMS contacts.

In rare cases it can be physically impossible to use the Digistat system.

The healthcare organization using Digistat should define an emergency procedure to face this kind of events and to:

- 1) allow the hospital departments to be operational again;
- 2) restore the system to full availability as soon as possible.

Ascom UMS can provide support and assistance for the emergency procedure, but the healthcare organization is responsible for the procedure definition and execution.

See section 5 for the Ascom UMS contacts.

3. Digistat Instructions for Use

The Digistat system instructions for use are provided with the product. They can be displayed in digital format within the Digistat system (only this quick start guide is provided on paper). To access the instructions for use:

- Click the **Help** button on the command bar (Fig 8)



Fig 8

A screen opens, making it possible to access:

- 1) the relevant Digistat user manuals;
- 2) possible quick reference guides referring to specific procedures.

The online documentation is in PDF format.

3.1 On line update of the user manual

The Digistat user manual can also be downloaded from the internet. Every Digistat customer can access a protected area on which all the existing versions of the manuals are present. The user can this way check if an updated version of their manual exists. To download the Digistat manuals, type the following address on your web browser address bar:

<https://confluence.ascom-ws.com/display/DIG/DIGISTAT+EU+v6.0>

A welcome page is displayed. Enter username and password to access the User Manuals list.

Username – User name provided by Ascom UMS/Distributor

Password – Password provided by Ascom UMS /Distributor

4. How to display the Digistat about box

To display the Digistat about box

- Click the **Menu** button on the Control Bar (Fig 9 A)



Fig 9

A menu containing different options opens (Fig 10).

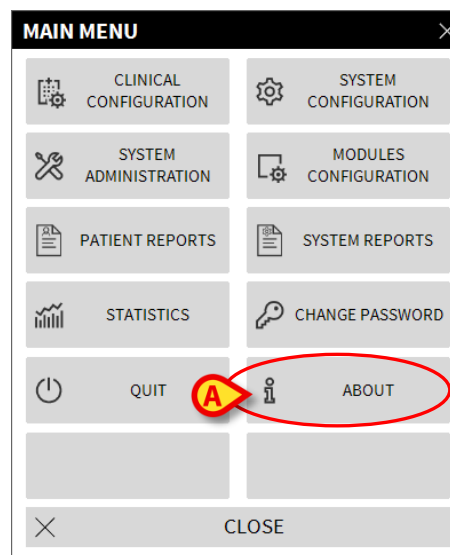


Fig 10

- Click, on the menu, the **About** button (Fig 10 A). The Digistat about box opens (Fig 11).



Fig 11

4.1 Printed Instructions for Use

Customers can request a printed version of the user manuals for the purchased products. Ascom UMS will provide the requested user manuals as soon as possible.

The request must be forwarded to the technical assistance. See section 5 for the list of Ascom UMS contacts.

5. Contacts

For any issue, please refer first to the Distributor who installed the Product.

Here are the manufacturer contacts:

Ascom UMS s.r.l unipersonale

Via Amilcare Ponchielli 29, 50018, Scandicci (FI), Italy

Tel. (+39) 055 0512161

Fax (+39) 055 8290392

Technical assistance

support.it@ascom.com

800999715 (toll free, Italy only)

Sales and products information

it.sales@ascom.com

General info

it.info@ascom.com