ASCOM MYCO 4

Safety Instructions

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Sikkerhetsanvisninger

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Bezpečnostné pokyny

Turvaohjeet

Säkerhetsinstruktioner

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Precautions and Environmental Requirements

Frequency Range and Output Power

English	5
Dansk.....	9
Deutsch.....	11
Dutch	13
Español	15
Français.....	17
Italiano.....	21
Norsk	23
Português	25
Polski	27
Slovensky	29
Suomi.....	31
Svenska.....	33
Türkçe	35
Appendix 1.....	37
Appendix 2.....	39

English

Safety Instructions

Ascom Myco 4 Handset

Intended use

Your Ascom Myco 4 is intended to be a handset running compatible general purpose apps downloaded from any App Store.

Ascom Myco 4 is intended to be used as a hub for sharing information as well as reading, typing, and analyzing information, for example pictures and information via third-party apps installed on the handset.

Basic functionality of the Ascom Myco 4 handset is described in the Quick Reference Guide available at www.ascom.com/eifu

NOTE: Product designator for regulatory purposes of Ascom Myco 4 is SH4.

Safety Instructions

For safe and efficient operation of the handset, observe all necessary safety precautions when using the handset, see *Appendix 1 Precautions and Environmental Requirements*.

Privacy of communications may not be ensured when using this handset.

This product shall only be used with the following batteries:

P/N: 660733

Battery pack Ascom Myco 4

CAUTION: Risk of fire explosion if the battery is replaced by an incorrect type.

The handsets are powered by a rechargeable battery and shall be only charged using Ascom chargers.

Available chargers:

P/N: CB1-U1

Charging rack base

P/N: CTH-U01

Charger module for handset

P/N: CTH-U02

Charger module for handset and battery

P/N: CTB-U01

Charger module for battery

Backward compatible chargers:

P/N: DC5-AAAB

Desktop charger

P/N: CR7-AAAA

Charging rack

P/N: CR8-AAAA*

Battery pack charger

* + P/N 660746

Available power adapter and USB-C cable (suppliers designation):

P/N: 660736	USB power supply 18W
P/N: 660737	USB-C cable

Information about frequency bands and associated maximum output power is found in the *Appendix 2 Frequency Range and Output Power*.

Declaration of Conformity (EU/EFTA)

Hereby, Ascom (Sweden) AB declares that SH4 is in compliance with the essential requirements and other relevant provisions of the directives 2014/53/EU (RED) and 2015/863 (RoHS3) amending the directive 2011/65/EU.

The complete Declaration of Conformity is available at:
www.ascom-ws.com/doc/

The product is marked with the labels:  

Radio Frequency (RF) Exposure Information

The specific absorption rate (SAR) limit of Europe is 2.0 W/kg. This handset has been tested against this SAR limit. This handset was tested for typical body-worn operations with the back of the device kept 5 mm from the body. To maintain compliance with RF exposure requirements, use accessories that maintain a 5 mm separation distance between the user's body and the back of the handset. The use of belt clips, holsters and similar accessories should not contain metallic components in its assembly. The use of accessories that do not satisfy these requirements may not comply with RF exposure requirements, and should be avoided.

Declaration of Conformity (UK)

Hereby, Ascom (Sweden) AB declares that SH4 is in compliance with the essential requirements and other relevant provisions of Radio Equipment Regulations 2017 and the Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012.

The complete Declaration of Conformity is available at:
www.ascom-ws.com/doc/

The product is marked with the labels:  

Declaration of Conformity (USA and Canada)

This handset meets the applicable Innovation, Science and Economic Development Canada technical specifications.

Use of non-Ascom approved accessories may violate the FCC and ISED guidelines for RF exposure and should be avoided.

This handset has been tested and found to comply with the

limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This handset generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this handset does cause harmful interference to radio or television reception, which can be determined by turning the handset off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient, or relocate the receiving antennas.
- Increase the separation between the handset and receiver.
- Connect the handset into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer, or an experienced radio/TV technician for help.

CAN ICES-003 (B)/ NMB-003 (B)

Information to User

This handset complies with Part 15 of the FCC rules and with Industry Canada licence-exempt RSS standard(s).

Operation is subject to the following two conditions:

1. this handset may not cause harmful interference, and
2. this handset must accept any interference received, including interference that may cause undesired operation.

While operating in WiFi-6E (RLAN handsets in the 5925-7125 MHz band):

- This handset shall not be used for control of or communications with unmanned aircraft systems;
- This handset shall not be used on oil platforms;
- This handset shall not be used on aircrafts;
- This handset shall not be used on automobiles;
- This handset shall not be used on trains;
- This handset shall not be used on maritime vessels.

The handset is restricted to indoor use only when operating in the 5150 to 5250 MHz frequency range.

RF Exposure Information

The handset has been tested and found to comply with FCC RF radiation exposure limits set forth for an uncontrolled environment and meets the FCC RF exposure guidelines.

This handset is designed and manufactured not to exceed the emission limits for exposure to RF energy set by the Federal Communications Commission of the United States.

During SAR testing, this handset was set to transmit at its

highest certified power level in all tested frequency bands, and placed in positions that simulate RF exposure in usage against the head with no separation, and body-worn with no separation for Wi-Fi versions (SH4-Axxx) and 10mm separation for DECT versions (SH4-xxDx).

Although the SAR is determined at the highest certified power level, the actual SAR level of the handset while operating can be well below the maximum value.

The exposure standard for wireless devices employing a unit of measurement is known as the specific absorption rate, or SAR. The SAR limit set by the FCC is 1.6 W/kg.

The FCC has granted an equipment authorization for this handset model with all reported SAR levels evaluated as in compliance with the FCC RF exposure guidelines. This handset is compliance with SAR for general population/uncontrolled exposure limits in IC RSS-102 and had been tested in accordance with the measurement methods and procedures specified in IEEE 1528 and IEC 62209. This handset and its antenna(s) must not be co-located or operating in conjunction with any other antenna or transmitter.

For information about SAR values, see the Data Sheet for the handset.

Modifications

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the handset.

California Proposition 65

WARNING: This product can expose you to chemicals including Antimony trioxide, which is known to the State of California to cause cancer, and Di-isodecyl Phthalate (DIDP), which is known to the State of California to cause birth defects or other reproductive harm.

For more information go to www.P65Warnings.ca.gov

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Handset

Prevention of Hearing Loss

To prevent possible hearing damage, do not listen at high volume levels for long periods.

- Low volume levels might result in missed alerts.
- Only use the handset in temperatures between: -20° C and +60° C (-4° F and 140° F).
- Avoid exposing the handset to direct sunlight, heat sources, or moisture.
- Do not place a cold handset in a charger.
- Avoid sudden temperature changes to prevent condensation in the handset. It's recommended to put the handset in an airtight plastic bag until the temperature is adjusted, for example, when entering, or leaving, a cold/heated building on a warm/cold day.
- If the handset has been exposed or come into contact with water or condensate, remove the battery immediately and let it dry completely before reinserting the battery.
- Do not expose the handset to an open flame.
- Protect the handset from aggressive liquids and vapors.
- Remove the handset from the charger before cleaning the handset to reduce risk of an electric shock.
- Keep the handset away from strong electromagnetic fields.
- The handset may retain small metal objects around the earpiece region.
- Do not place heavy objects on the handset.
- Do not allow children to play with the product packaging material. This could cause choking and/or suffocation.
- Do not allow children to play with the handset. It's not a toy.

Battery

- Do not immerse the battery into water. This could shortcircuit and damage the battery.
- Do not expose the battery to an open flame. This could cause the battery to explode.
- Do not allow the metal contacts on the battery to touch another metal object. This could short-circuit and damage the battery.
- Do not leave the battery where it could be subjected to extremely high temperatures, such as inside a car on a hot day.
- Use the compatible chargers or the USB-C cable (together with the USB power supply) for charging.
- Charge the battery for at least one hour the first time you use the battery.
- Do not charge the battery when the ambient room temperature is below 5° C or above 40° C (below 41° F or above

104° F).

- Do not attempt to take the battery apart.
- Turn off the handset before removing the battery (recommended).
- Defective batteries must be returned to a collection point for chemical waste disposal.

Appendix 2

Frequency Range and Output Power

Wi-Fi and Bluetooth Frequencies

EU/EFTA	
Frequency Bands (MHz) and Channels	Max Output Power (dBm)
2400–2483.5 (Ch 1–13)	20
5150–5250* (Ch 36–48)	23
5250–5350* (Ch 50–64)	20
5470–5725 (Ch 100–140)	20
5725–5850 (Ch 149–165)	14
5945–6425 (Ch 1–93)**	23

USA	
Frequency Bands (MHz) and Channels	Max Output Power (dBm)
2402–2480 (Ch 0–78)***	9.5
2412–2462 (Ch 1–11)	23
5150–5250 (Ch 36–50)	20
5250–5350 (Ch 52–64)	22.5
5470–5725 (Ch 100–140)	22.5
5745–5850 (Ch 149–165)	22
5925–7125 (Ch 1–233)**	17.5

Canada	
Frequency Bands (MHz) and Channels	Max Output Power (dBm)
2402–2480 (Ch 0–78)***	9.5
2412–2462 (Ch 1–11)	23
5150–5250 (Ch 36–50)	20
5250–5350 (Ch 52–64)	22.5
5470–5725 (Ch 100–140)	22.5
5745–5850 (Ch 149–165)	22
5925–7125 (Ch 1–233)**	17.5

* Indoor use only.

** Wi-Fi 6e and indoor use only.

*** Bluetooth frequency.

Additional National Restrictions (Canada)

The 5600–5650 MHz frequency band is disabled for use.

Bluetooth

Radio Spectrum (MHz)	Max Output Power (dBm)
ISM 2400–2483.5	Class 1, maximum 20 dBm

Near Field Communication

NFC Band (MHz)	Field strength (dBμA/m @10m)
13.56	42

Cellular Frequencies

Note: Only cellular versions SH4-xxCx

LTE	
Frequency Bands (MHz) and Channels	Max Output Power (dBm)
LTE band 1	25.7
LTE band 2	25.7
LTE band 3	25.7
LTE band 5	25.7
LTE band 7	25.7
LTE band 8	25.7
LTE band 20	25.7
LTE band 26	25.7
LTE band 28	25.7
LTE band 38	25.7
LTE band 40	25.7
LTE band 41	25.7

GSM	
Frequency Bands (MHz) and Channels	Max Output Power (dBm)
GSM 850	35
GSM 900	35
GSM 1800	32
GSM 1900	32

WCDMA	
Frequency Bands (MHz) and Channels	Max Output Power (dBm)
WCDMA Band 1	25.7
WCDMA Band 2	25.7
WCDMA Band 5	25.7
WCDMA Band 8	25.7

5G	
Frequency Bands (MHz) and Channels	Max Output Power (dBm)
band n1	25.7
band n3	25.7
band n5	25.7
band n7	25.7
band n8	25.7
band n20	25.7
band n28	25.7
band n38	25.7
band n40	25.7
band n41	25.7
band n77	25.7
band n78	29.0

DECT Frequencies

Note: Only DECT versions SH4-xxDx

Region	DECT Allocation (MHz)	Max Output Power (mW)
Europe	1880-1900	250
USA	1920-1930	100
Latin America	1910-1930 1910-1920 (Brazil)	250
Thailand	1900-1906	250

Manufactured by:

Ascom (Sweden) AB

Grimbodalen 2,
SE-417 49 Göteborg, Sweden
www.ascom.com

US importer:

Ascom (US), Inc.
300 Perimeter Park Dr, Morrisville, NC 27560

PM000353 / Ver. J / December 2024